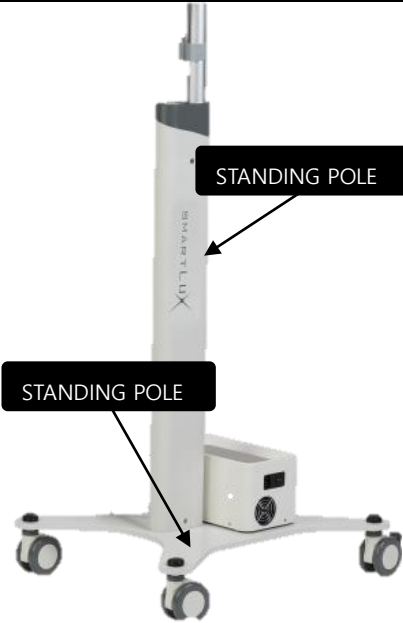
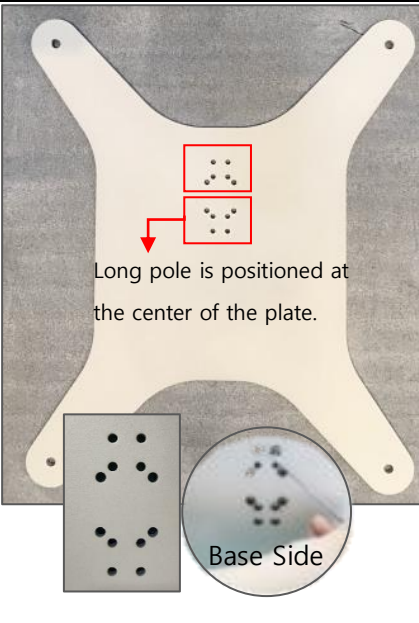



SMARTLUX MINI_ASSEMBLING MANUAL

<p>[1] Mount the Standing Pole on the Base Plate as below pic.</p>	<p>[2] To fix the Standing Pole to the Plate, screw 3mm bolts X 8EA by Allen wrench.</p>	<p>[3] Fit Support Ring first into the pole and then put the Hold Bracket.</p>
		
<p>[4] Insert Molex Cable into the pole housing as below pic. ①Insert the cable through the hole of the pole housing. ②Pull out the cable from the top of the pole housing.</p>	<p>[5] Set Arm on the standing pole. *Pay attention to the arm direction! *The groove of the arm should fit on the prominent part of the hold bracket.</p>	<p>[6] Complete *Arrange the molex cable and power cable as below pic.</p>
